




THERABATH®

soothing touch • renewing life

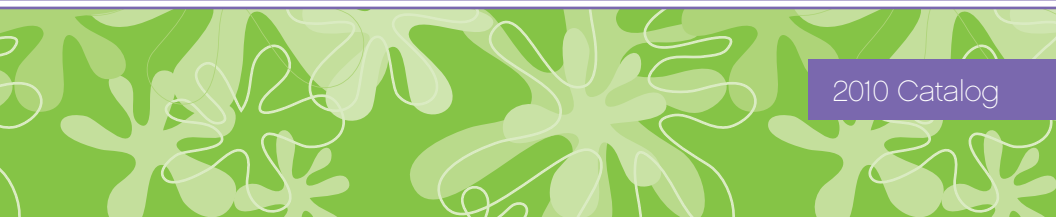

2010 Catalog



Because Life is Most Enjoyed Hands-On!

Therabath is dedicated to helping you enjoy life to it's fullest.

Since 1962, the Therabath Professional Paraffin Wax Bath has proudly been made to professional healthcare standards right here in the USA, but is available world-wide. We make sure each unit that leaves our factory is **durable, safe, and reliable**. We provide quality products and service because when you're in pain, you want **fast, dependable relief**.

The Therabath provides that relief through moist heat therapy. Professionals recommend paraffin treatments for relieving chronic pain during rehabilitation, while non-professionals use it to soothe aches and pains, and to keep skin soft and healthy looking.

Whether you use the Therabath for pain relief, dry skin, or just soothing warmth, it will leave you relaxed, renewed, and ready to **enjoy your life, hands-on!**



Instant Warmth – Radiant Softness

Benefits of Paraffin Therapy

Medical

- Provides therapeutic relief of pain due to arthritis, joint inflammation, muscle stiffness, and injury.
- Provides Instant warmth
- Increases circulation to renew tissue
- Feels soothing and relaxing

Beauty

- Topically moisturizes skin
- Plumps and hydrates to minimize appearance of fine lines and wrinkles
- Opens pores to detoxify
- Leaves skin soft and healthy looking



INDICATIONS:

Use for symptomatic relief of pain due to arthritis, bursitis, chronic joint inflammation, muscle stiffness, and muscle spasms; for rehabilitation after fractures, sprains, and injuries; and to loosen muscles and joints prior to exercise and massage. Consult your physician if you have questions.

CONTRAINDICATIONS:

Do not use in presence of open wounds, inflammatory skin conditions, certain peripheral vascular diseases, acute inflammation, or when sensation of extremity is reduced as in some cases of diabetes. Do not allow children under 12 to use unit. If there is any question about peripheral vascular disease or decreased sensation of extremities, consult a physician before using. Please see complete Indications and Contraindications in Operators Manual or website, www.therabath.com.

...feel the
difference!



The Therabath Difference

Quality Workmanship, Quality Materials

- Made in the **U.S.A** since 1962
- Reliable **24/7** operation
- A **flame retardant** outer case
- A **coated-steel, oversized, inner tank** fits up to a men's size 14 (US) foot
- Two worry-free **automatic thermostats** safely maintain a therapeutic temperature range of 126 to 134° f (52.2 - 56.7°c)
- **Energy efficient**, fully insulated
- A **hospital grade electrical cord**

Unit dimensions: 14" X 8" X 8", (36 X 20 X 20 cm)

Unit weight (full): 15lbs (6.81 kg)

"I was hurting so bad...

I needed help managing the swelling and discomfort I was having in my hands. Due to my allergies I sought out the most natural way I could find to get relief. With the help of my doctor, as a team, we discussed paraffin therapy. As a benefit, I could do this at home. It was more cost effective than multiple trips to the rehab center. *Paraffin therapy was the best choice for me!*"



MADE IN
THE USA
SINCE 1962



GMP

Good Manufacturing Practice

Therabath uses superior materials and manufacturing practices for the **highest quality bath available.**



UNSURPASSED LIFETIME GUARANTEE!

WR Medical Electronics Co. warrants the Therabath® to be free from defects in material and workmanship under normal use and service as long as the original purchaser owns the product. See full warranty in the Operators Manual or at www.therabath.com for complete details. This limited Lifetime Guarantee assures your complete satisfaction. No other manufacturer matches our promise of quality!

Professional PARAFFIN BATH

LIFETIME
GUARANTEE!



THERABATH PROFESSIONAL PARAFFIN BATH

	Model TB5 - 110 Volt	Model TB4 - 220 Volt	Paraffin Scent
Item#	2300	2302	Wintergreen
	2320	2322	ScentFree™
	2310	2312	PeachETM
	2330	2332	Lavender Harmony™
	2340	2342	Cucumber Melon
	2350	2352	Orange Vanilla
	2375	2373	eo Fresh Squeezed Lemon
	2378	2379	eo Lilac & Lavender
	2383	2382	Vanilla Cupcake
	2386	2387	eo Rose Petal
NEW!	2384	2385	Fresh Rain (aloe, vitamin e & jojoba oils)
SEASONAL	2389 / 2370	2390 / 2372	eo Warm Apple Spice / Chocolate Mint

eo : features therapeutic grade essential oils

www.therabath.com

See, Smell, & Feel the Difference!

Aromatherapy paraffin renews the body & spirit.

Experience the most luxurious paraffin available - **see** Therabath Refill Paraffin's crystal clear purity, **feel** its silky softness, and **smell** its delicate scent. Just a few dips result in a thick and pliable paraffin glove that warms and relaxes, making pain and dry skin a distant memory. Though best appreciated in a Therabath Professional Paraffin Bath, Therabath Refill Paraffin can be used in any paraffin bath.

Therabath Refill Paraffin is:

- FDA food grade
- Triple filtered for purity
- Crystal clear
- Soft, pliable, and non-greasy
- Packed in easy-open 1 lb. (0.454 kg) bags of paraffin beads



Sample our scents...

Not sure which scent fits your personality? Call us for a free half-ounce sample jar.



Scents to satisfy every personality...



Wintergreen
refreshingly minty



ScentFree™
dye & fragrance free,
hypoallergenic



PeachE™
soft, delicious peach



Lavender Harmony™
fragrant, relaxing



Cucumber Melon
crisp, calming and refreshing



Orange Vanilla
creamy, juicy, ice cream desert



Fresh Rain
fresh, clean and moisturizing
Infused with aloe, vitamin e and
jojoba oils



Rose Petal
sweet and romantic
Infused with rose & geranium
essential oils



Fresh Squeezed Lemon
clean, uplifting citrus
Infused with lemon essential oil



Lilac & Lavender
fresh blooming lilacs
Infused with lavender essential oil



Vanilla Cupcake
like stepping into a bakery



Warm Apple Spice
juicy apples, fresh spices & pear
Infused with clove & cinnamon
essential oils



Chocolate Mint
warm brownies, zero calories!
Infused with peppermint essential oil

Call us to create your own custom scent!

THERABATH REFILL PARAFFIN

6lb. carton (2.72 kg)	24lb. box (10.89 kg)	Paraffin Scent
0100	0150	Wintergreen
0101	0151	ScentFree™
0102	0152	PeachE™
0107	0153	Lavender Harmony™
0122	0154	Cucumber Melon
0124	0155	Orange Vanilla
0130	0159	eo Fresh Squeezed Lemon
0134	0161	eo Lilac & Lavender
0138	0163	Vanilla Cupcake
0132	0160	eo Rose Petal
NEW! 0140	0164	Fresh Rain (aloe, vitamin e & jojoba oils)
SEASONAL 0136 / 0128	0162 / 0158	eo Warm Apple Spice / Chocolate Mint

eo : features therapeutic grade essential oils

Enhance Therabath ParaFFin Treatments

Convenient kits include all the items needed for the most effective hand, foot or face paraffin treatment.



Hand ComfortKit™

2 Insulated Terry Mitts
100 Liners
4 oz. Hydrating Cream
4 oz. Sanitizing Spray

item # 2400



Foot ComfortKit™

2 Insulated Terry Boots
100 Liners
4 oz. Hydrating Cream
4 oz. Sanitizing Spray

item # 2401



(a,b) Sanitizing Spray; (c-e) 4 oz. Paraffin Treatment Cream; (f) 8 oz. Paraffin Treatment Cream



Facial Treatment Kit

50 Gauze Masks
4 oz. Hydrating Cream
1" Facial Brush

Facial Kit - item # 2402

50 Masks - item # 2412

"A LA CARTE" ACCESSORIES



Mitt/Boot Liners - item # 2420

100 pack, durable, disposable.
Case of 10 pkgs - item #2421.



item #	description
2410 / 2411	Insulated Terry Velour Mitts / Boots, pair 8 oz. fiber-fill insulation, 100% natural cotton shell, machine wash, hook/loop closure.
1307 / 1308	Mitt Kit / Boot Kit - 2 insulated mitts or boots and 100 plastic liners
0205 / 0206	Paraffin Treatment Cream, 4 oz. / 8 oz (scentfree, vanilla cupcake, or lavender harmony) for skin and nails with oils of lavender, camphor, & peppermint.
2432	Sanitizing Spray, 8 oz. - for skin and nails with oils of lavender, camphor, & peppermint.
2207 / 2206	Application Strips, 50 pack / 250 pack
2209	Facial Application Brush - 1 inch
2210 / 2213	Body Application Brush - 2 inch / 3 inch



therabath quick tips

body

1. Place the Therabath as close to work area as possible. Remove the number of strips you think you will use from package (average 4-8 strips). Suggested areas include: elbows, knees, lower leg/calf, thigh, forearm, back, neck or shoulders.
2. Holding one end of strip, dip entire strip in paraffin and remove. Hold over Therabath until paraffin stops dripping.
3. Wrap area to be treated, stretching strips slightly. Apply additional strips, wrapping and overlapping by half so area has at least 2 layers of paraffin coated strips. The more strips, the more heat and more relief.
4. Leave strips in place for at least 10 minutes and then remove by massaging them slightly and sliding them off.
5. Discard used strips and repeat as necessary.



APPLICATION TOOLS



Application Strips
50 pack or case of 250.
Stretchy and disposable.
Strips are 3.25" w x 17.25" long.
Made in the U.S.A.

Strips - item # 2206 / 2207



Application Brushes
Natural bristles. Feature
rubberized soft-grip handle.
Sizes: 1" facial, 2" or 3" body
Made in the U.S.A.

Brush Set - item # 2214



therabath quick tips

hands



1. Cleanse hands and mist with **Sanitizing Spray**. Rub in.
2. Apply a light coat of **Intensive Hydrating Cream**.
3. Spread fingers slightly, slowly immerse hand into paraffin and remove hand from paraffin
4. Wait 5-7 seconds, dip again and remove.
5. Repeat 3-5 times
 - the objective is to add several layers of paraffin to make a warm paraffin "glove"
 - the more paraffin, the longer the heat relief
6. Cover hand with plastic **mitt liner bag** and **terry mitt**
7. After 10 minutes, remove mitt, move hands and fingers to loosen paraffin and remove it in liner bag
8. Discard used paraffin.



For information regarding hygiene of paraffin treatments, visit www.therabath.com



therabath quick tips

feet



1. Cleanse feet and mist with **Sanitizing Spray**. Rub in.
2. Apply a light coat of **Intensive Hydrating Cream**.
3. Slowly immerse foot into paraffin and remove
4. Wait 5-7 seconds, dip again and remove.
5. Repeat 3-5 times (**NOTE:** feet are generally more sensitive to heat than hands – dip only portion of the foot on first dip, and the entire foot on subsequent dips)
6. Cover foot with plastic **boot liner bag** and **terry boot**
7. After 10 minutes, remove boot, move foot and toes to loosen paraffin and remove it in liner bag
8. Discard used paraffin.



therabath quick tips

face


1. Place facial brush in unit for 5 minutes to soften
2. Use a headband to hold back hair, remove all make-up, cleanse skin, and place cotton pads over eyes
3. Starting at forehead, brush a thin layer of paraffin on face & neck with a **paraffin application brush**.
4. Hold the edges of a pre-cut **gauze facial mask** with both hands, dip mask completely in & out of bath, and allow excess wax to drip off.
5. Place coated mask on face and quickly brush on 3-4 additional layers of wax on mask.
6. Relax for 15 minutes! Remove mask by peeling gently from the neck up.
7. Apply a light coat of **intensive hydrating cream** (optional) and allow skin to cool before applying makeup.
8. Clean application brush by soaking in bath for 5 min. Remove & wipe clean with paper towel.



therabath quick tips

cleaning

1. Clean unit when you see sediment has built up on bottom of inner tank.
2. Unplug unit, remove lid, and grille and allow paraffin to harden overnight.
3. The next day, plug in unit for approximately 10 minutes to melt edges of hard paraffin "cake" around the inside of the tank. Lay out paper towels to set wax "cake" on.
4. Press down on one end of the "cake" until it slides out the other end of the unit. Lift out "cake" and place on paper towel, flat side down.
5. If you want to reuse the clean paraffin, shave off the top quarter-inch, which will appear dirty, and discard.
6. Blot up remaining paraffin in tank with a soft paper towel and wipe unit dry. **HINT:** a hair dryer will melt hard paraffin drips, then blot them up with a paper towel
7. If you are reusing the paraffin, replace the "cake" in the unit, place the grille on top of paraffin, and replace lid.
8. Plug in the unit.
9. After paraffin has melted (8-10 hours), add additional wax as necessary to bring up to required level.

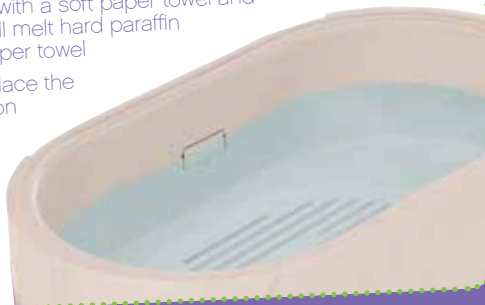



Relief for Work

“ I use my hands 8 hours a day as a Dental Professional. After 36 years of repetitive movements, osteoarthritis has caused me pain and stiffness. Paraffin gives me some relief and allows me to continue working daily. ”



Aahhh!

“ Paraffin treatments in therapy are a highlight of the visit! In earlier years I thought of it as a luxury for home use. Now, it's pain control for my days after gardening or needlework. Aahhh! Better than any pain killer. ”



WR MEDICAL ELECTRONICS CO.

123 North Second Street
Stillwater, MN 55082 USA

651-430-1200 | toll Free 800-321-6387 (USA only)

Fax 651-439-9733

therabath@wrmed.com

Copyright © 2008-2010 by WR Medical Electronics Co. All rights reserved.
Therabath is a registered trademark of WR Medical Electronics Co. Lit # MRI05, Rev 09/09

www.therabath.com